Art.260216

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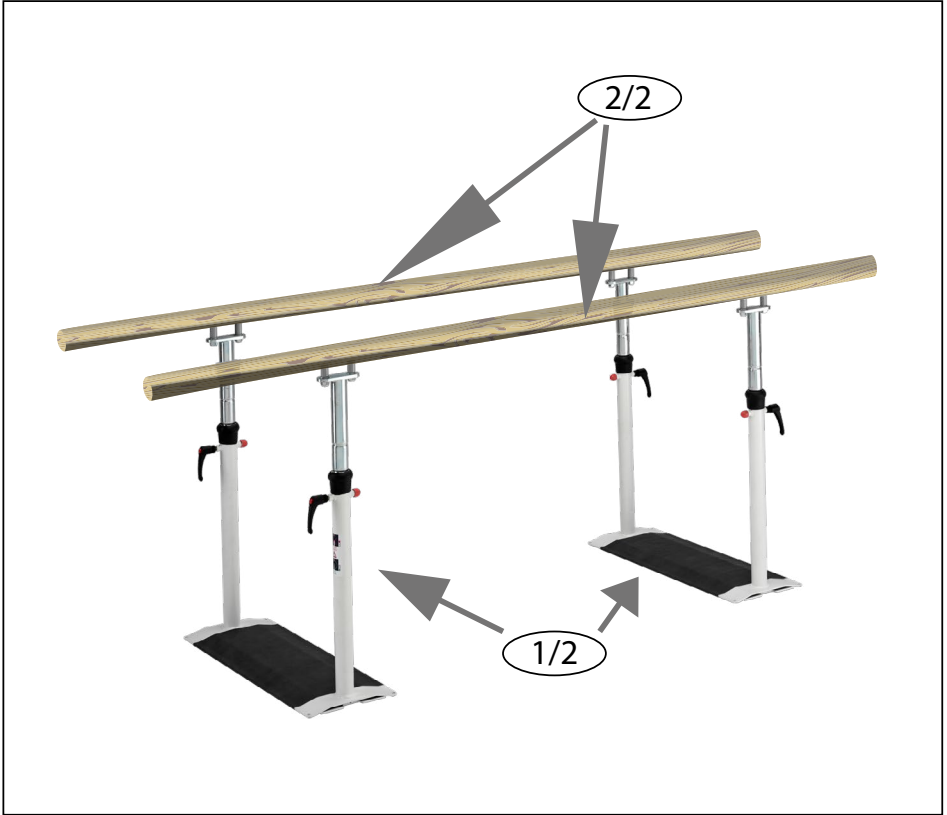
IT IS IMPERATIVE TO FOLLOW THE INSTRUCTIONS BELOW TO ASSURE CORRECT ASSEMBLY, INSTALLATION AS WELL AS CORRECT USE OF THE PARALLEL BARS 260216 OBLIGATION TO ANCHOR TO THE FLOOR, ACCORDING TO DIN EN 12182.

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
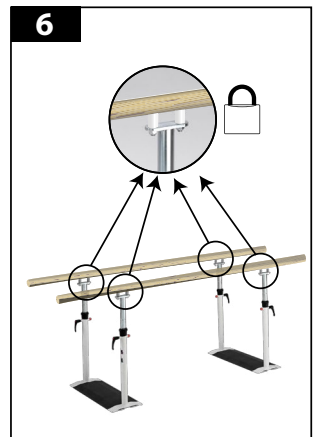
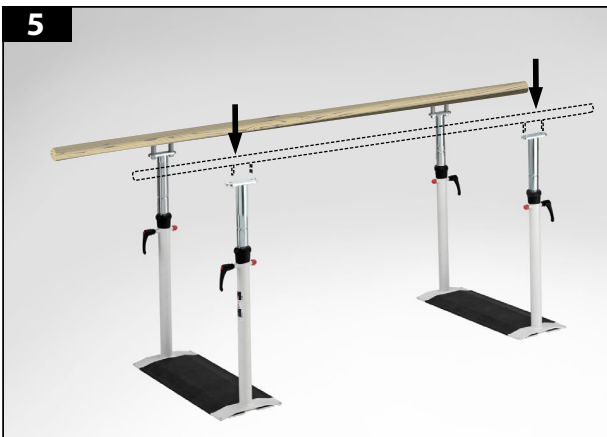
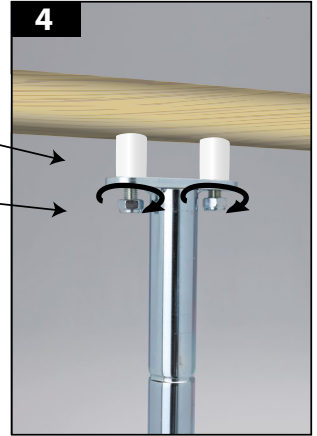
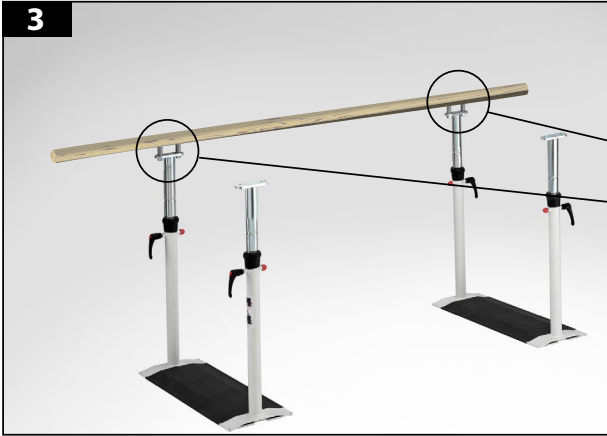
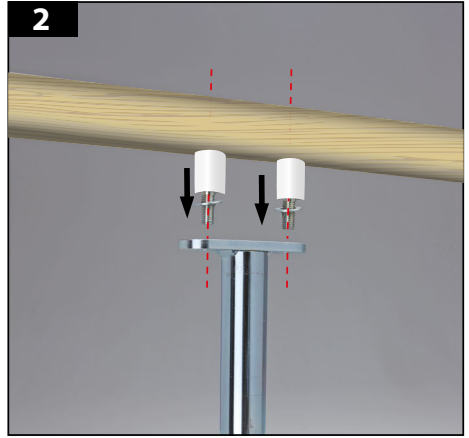
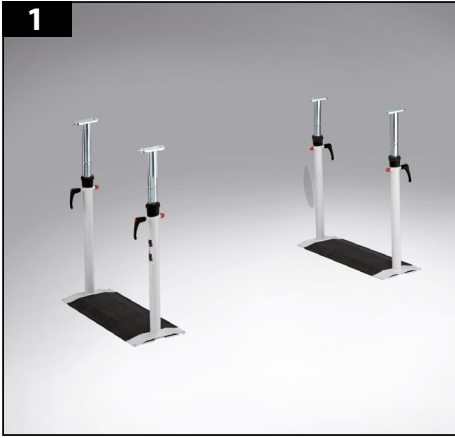
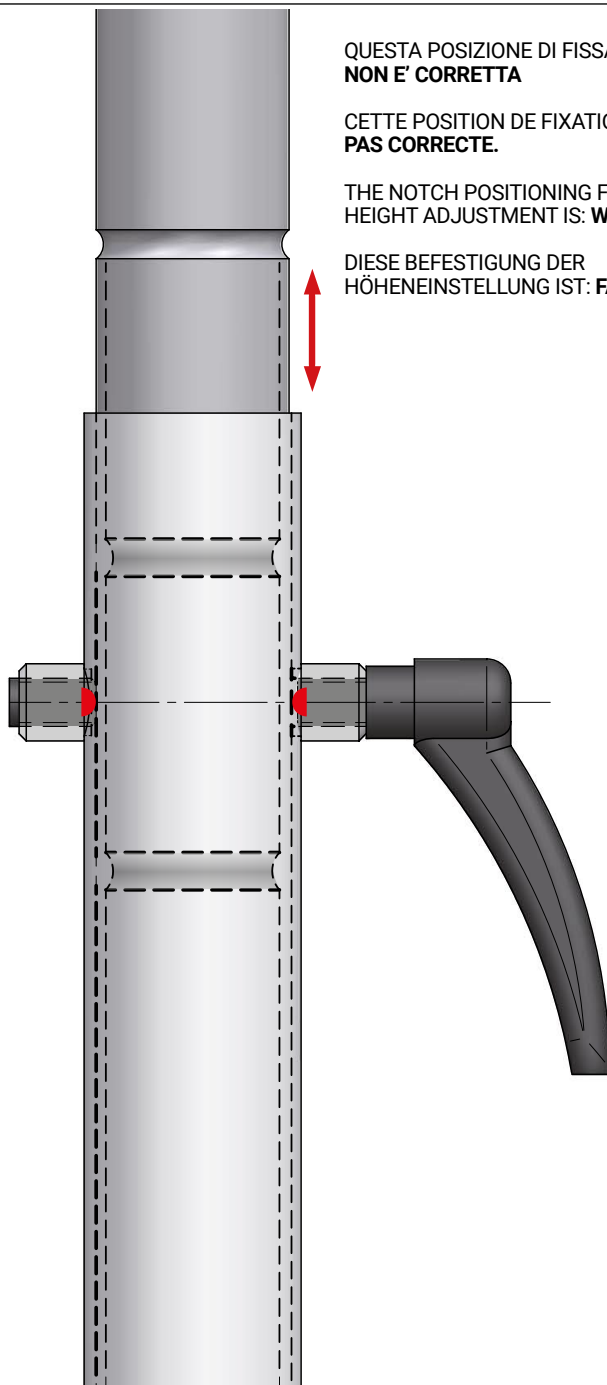
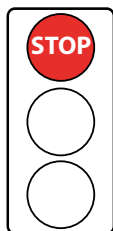
THE NOTCH POSITIONING FOR
HEIGHT ADJUSTMENT IS: **WRONG.**

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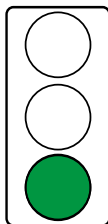·THE HEIGHT OF THE PARALLEL BARS IS CORRECTLY AND SECURELY ADJUSTED, ONCE THE FASTENER TIGHTLY SITS IN ONE OF THE SPECIFICALLY DESIGNED NOTCHES.

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·THE NOTCH POSITIONING FOR HEIGHT ADJUSTMENT IS: **CORRECT.**
IN THIS ALIGNMENT THE FASTENER (A) IS CORRECTLY POSITIONED TO SNAP IN **THE DESIGNATED** NOTCH.

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
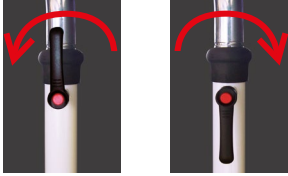
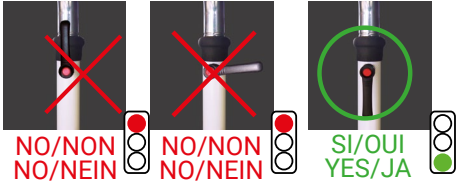
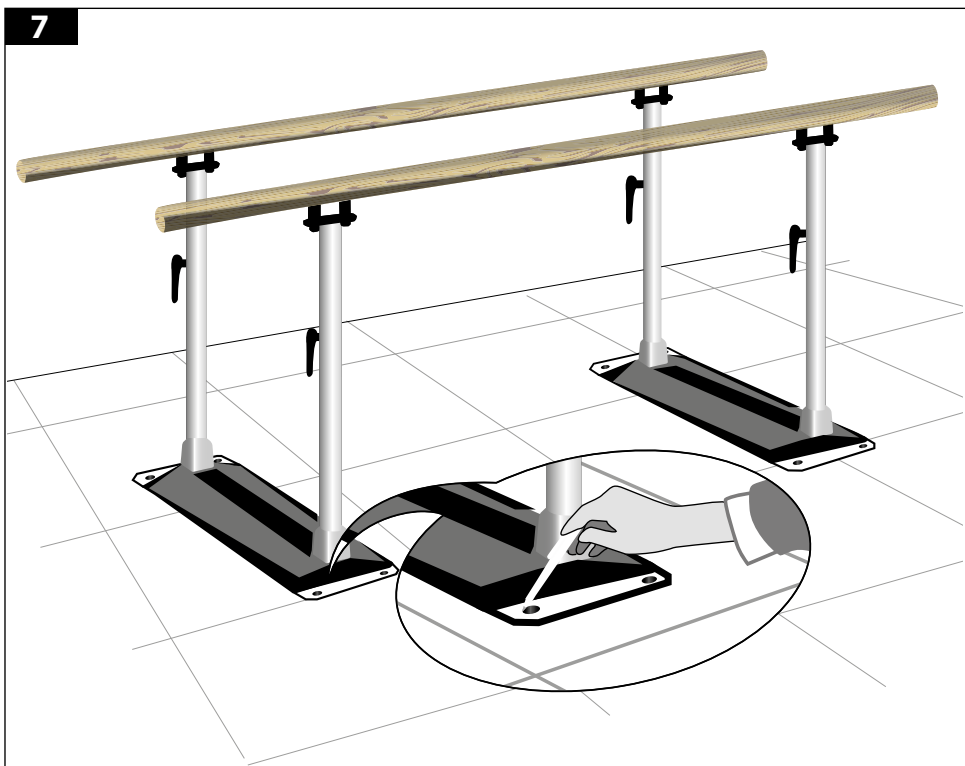
 **Obligation to fix to the floor, as required by UNI EN 12182**

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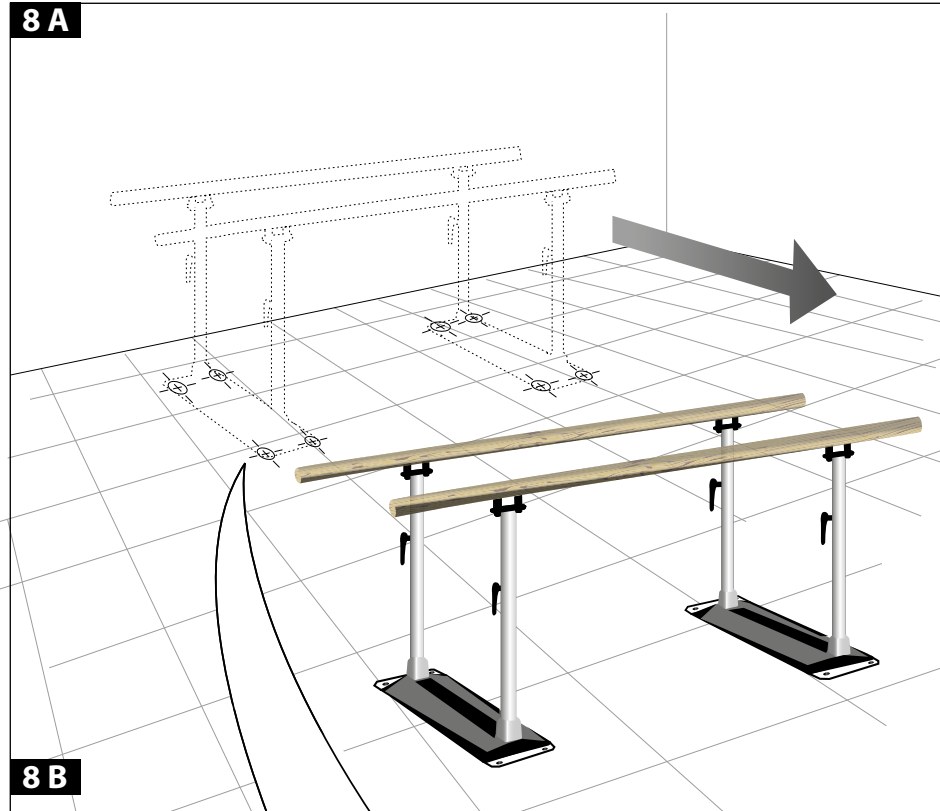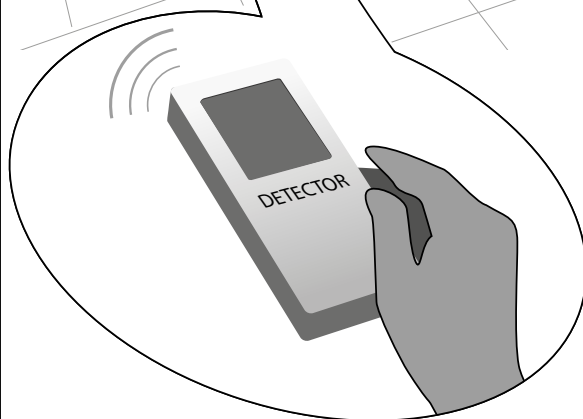
• Before drilling use a detector to check for pipes or electrical cabling in the wall.

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C. F. - P. IVA - R. I. IT 03613220262

✉ info@ferrox.it

Under its sole responsibility state that the products:

Name	Code	Identified by UDI-DI:
PARALLEL BARS "EASY CLICK" KLASSIC	260216.PN	(01)08053470432602164U

meet all the essential applicable essential requirements, of Annex I, implemented by the
(UE) REGULATION 2017/745 OF THE EUROPEAN PARLAMENT AND COUNCIL on Apr 5 th 2017.

Classified (Annex VIII, MDR 2017/745):

Class I (Rule 1)

Procedure for the assessment of conformity:

Technical documentation as per annex II and III.

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FOLDING PARALLEL BARS EASY CLICK: 260215, 260216, 260217, 260218, 260219

INTENDED USE

Foldable parallel bars for rehabilitation use.
For patients immobilised by simple fractures, motor recovery after minor trauma.

TYPE OF PATIENT

Adults who need short periods of walking rehabilitation.

PLACE OF USE

Medical clinics with limited spaces, home.

OPERATORS QUALIFIED TO USE THE PRODUCT

All healthcare professionals, physiotherapists, and related industries.

The instructions given to the patient by the reference healthcare professional suffice.

CONTRAINDICATIONS

Follow the safety instructions in user manual.
Not suitable for sports use.

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