RECOVER COOLING SLEEVE

Cold therapy compression for arm and leg

EN A compression sleeve for cold therapy on sore muscles and joints. Wear ReCove on forearm, elbow, leg, knee, ankle or wrist for muscle recovery. Using ReCove gives a nice sensation when having inflammation, arthritis, swelling or tension. It's also nice to wear after a tough workout. The sleeve can be used self-cooled or frozen for a colder experience.

Cold therapy makes the blood vessels contract and blood flow decrease, which reduces swelling and counteracts inflammation.

- Use when experiencing inflammation, swelling, tension or pain in muscles and joints.
- When wearing ReCove, the cooling effect lasts for about 15 minutes.
- Keep the sleeve on for compression or remove it to self-cool for 30 minutes.
- For a colder effect place ReCove in the freezer (about 2 hours). Use the frozen sleeve maximum 15 minutes on bare skin.
- Always keep ReCove in the enclosed plastic bag. If not, the sleeve loses its function and softness.
- Use damp cloth to wipe it clean. Lay flat and air dry.

Support

Recovery

Cooling treatment  
15 minutes

Material: Fabric: Lycra 100%, Gel: Glycerol 60%, Water 26%, Sodium Acrylate 14%

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