SISSEL[®] SIT AIR

- ▮ Produktinformation
- ▮ Product information
- ▮ Information produit
- ▮ Productinformatie
- ▮ Información sobre el producto
- ▮ Informazioni sul prodotto
- ▮ Produktinformation

SISSEL® SIT AIR

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SISSEL® SIT AIR

COMFORTABLE SITTING WEDGE FOR ACTIVE AND ANATOMICALLY CORRECT SITTING WITH THE ADDED EXTRA OF WORKING BACK AND CORE MUSCLES.

Function / indication

The use of the SISSEL® SIT AIR combines the desired upright position of a sitting wedge with the dynamics of an instable inflatable pillow. It is a dynamic, multi-functional sitting wedge for use in rehabilitation, at home, at work or any place where the user wishes to sit in an anatomically correct and comfortable way. Using the SISSEL® SIT AIR not only makes the user sit upright, it also helps train his core and back musculature in an almost unnoticeable way.

The SISSEL® SIT AIR supports the correction of many medical indications. These include the treatment, relief and prevention of posture-related muscular imbalance and back pain as well as balance training. The density of the special foam features the right balance between support and comfort.

- I The tilted surface of the wedge encourages a correct posture. This posture relieves and corrects many posture-related problems and it takes pressure off the spine and intervertebral discs.
- I The special design features a recess, ensuring that the legs are slightly spread, which has a beneficial effect on posture and avoids unhealthy heat accumulation.
- I The center is slightly instable, which helps the user to unconsciously train muscles of back and abdomen. The degree of instability can be adjusted with the help of the integrated valve.

Contraindication

Patients with unexplained medical conditions or complaints, those with unstable fractures of the spine or pelvis etc.

How the sitting wedge works

- I Without a wedge: Difficult to sit up straight, tendency to sit hunched.
- I With a SISSEL® Sitting Wedge: The body is moved effortlessly into an upright position with the pelvis tilted slightly forward.
- I Imbalance: **1** The valve helps you adjust the degree of imbalance, however you like it.

Use

- I For daily use as a sitting support on a sturdy surface, i.e. chair, bench or stool.
Not suitable for use on soft furnishings, e.g. armchair or sofa.
- I Dynamic sitting under direct supervision or independently after instruction from medical professionals.

Material / care

- I The product features high-quality materials and solid workmanship.
The mesh cover promotes air circulation.
- I material: core: polyurethane foam, cover: 100% polyester mesh, adjustable valve
- I size: approx. 35,5 x 35 x 5,5 cm
- I Hand wash only. Air dry, do not dry on radiators or in direct sunlight (color will fade!).

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